To watch video instructions, please visit our website: www.sigvaris.com/videos

DND: Net weight: 700 gram / Gross weight: 840 gram / Box size (mm) LxWxH: 125 x 125 x 160 mm

CONE: Net weight: 470 gram / Gross weight: 720 gram / Box size (mm) LxWxH: 112 x 112 x 52 mm

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If you look for further distributors please consult: www.sigvaris.com

SIGVARIS and SIGVARIS GROUP are registered trademarks of SIGVARIS AG, CH-9014 St. Gallen/Switzerland, in many countries worldwide. © 2021 Copyright by SIGVARIS AG, CH-9014 St. Gallen/Switzerland

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EN: DOFF N' DONNER – doffing device for compression stockings:
This is how you can use the DND to remove compression stockings.

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- 1:** Place the DND over the heel of the stocking.
- 2:** Roll the DND down the length of the stocking.
- 3:** Push the stocking over the DND to remove it.

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DOFF N' DONNER PRODUCTS consisting of:
A: DND
B: CONE (sold separately)
Intended purpose: Aids donning and doffing of medical compression stockings
Contraindications: No known contraindications
Indications:
• Pain in the back, hands, arms or legs
• Rheumatism, arthritis, arthrosis
• Obesity
• Problems when bending
Suitable for the following models:
A-D Calf stocking, open / closed toe, with or without grip top
A-G Thigh stocking, open / closed toe, with or without grip top

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EN: The DOFF N' DONNER (DND) is a soft device that allows you to quickly and easily guide a compression sock or stocking up and down a leg or arm, making it simple to put on and remove all styles of medical compression. The DOFF N' DONNER CONE (CONE) simplifies the rolling of a compression stocking onto the DND. With the aid of the CONE, the medical compression stocking can also be easily unrolled from the DND.

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EN: Clamp the CONE to a smooth surface (press the lever down, CONE adheres firmly).

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EN: Ensure compression stocking is turned right side out. Place the compression stocking with the top band (soft top or grip top) over the CONE.

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EN: Roll the compression stocking completely over the CONE. **Closed toe:** Position the toe seam flat on the CONE tip. **Open toe:** Stocking should end 3 fingers width from the CONE tip.

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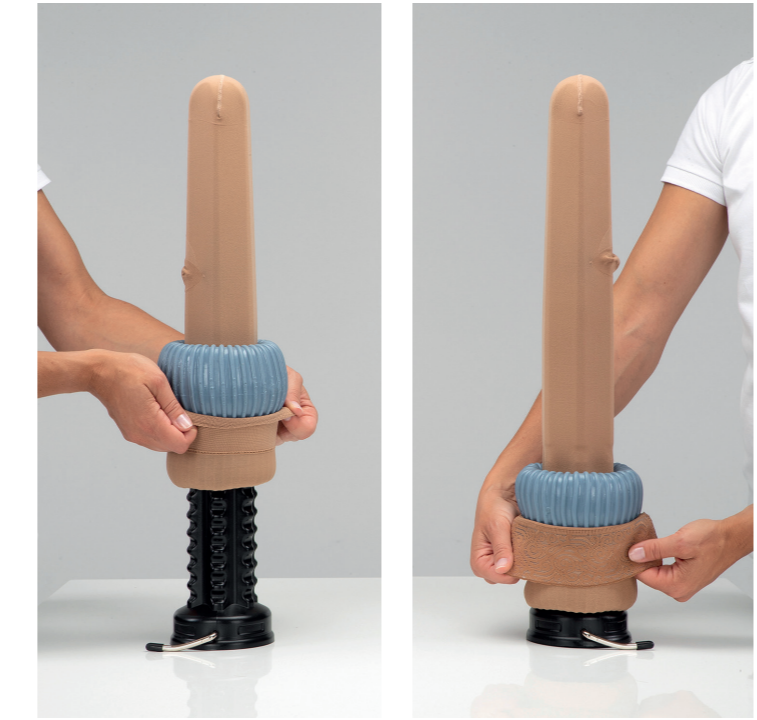
EN: Place the DND over the CONE and roll it downward. Pull the top band of the compression stocking up over the DND.

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EN: Calf stocking: Roll the DND upward until the compression stocking is rolled completely onto the DND.

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EN: Thigh stocking: Gently pull the top band of the compression stocking beyond the tip of the CONE to avoid bunching. Grasp the DND and roll it upward until the compression stocking is rolled completely onto the DND.

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EN: Gently roll the DND to flatten the toe seam. Hold the DND so the heel of the compression stocking is visible from the outside and is facing the floor.

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EN: Place the toe seam on the toes. The heel of the rolled-up compression stocking points downward. **Open toe:** Place the cuff according to the arrow.

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