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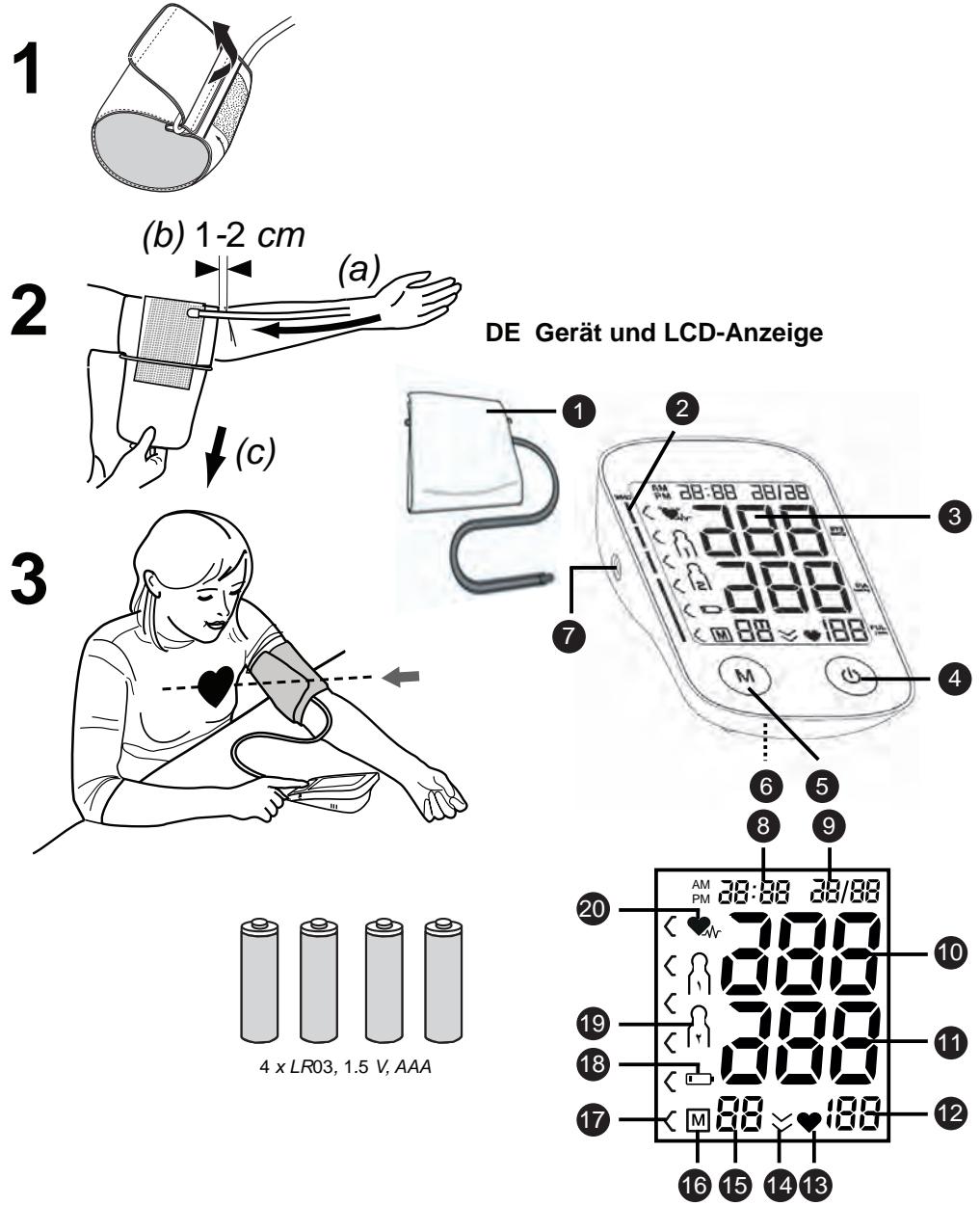
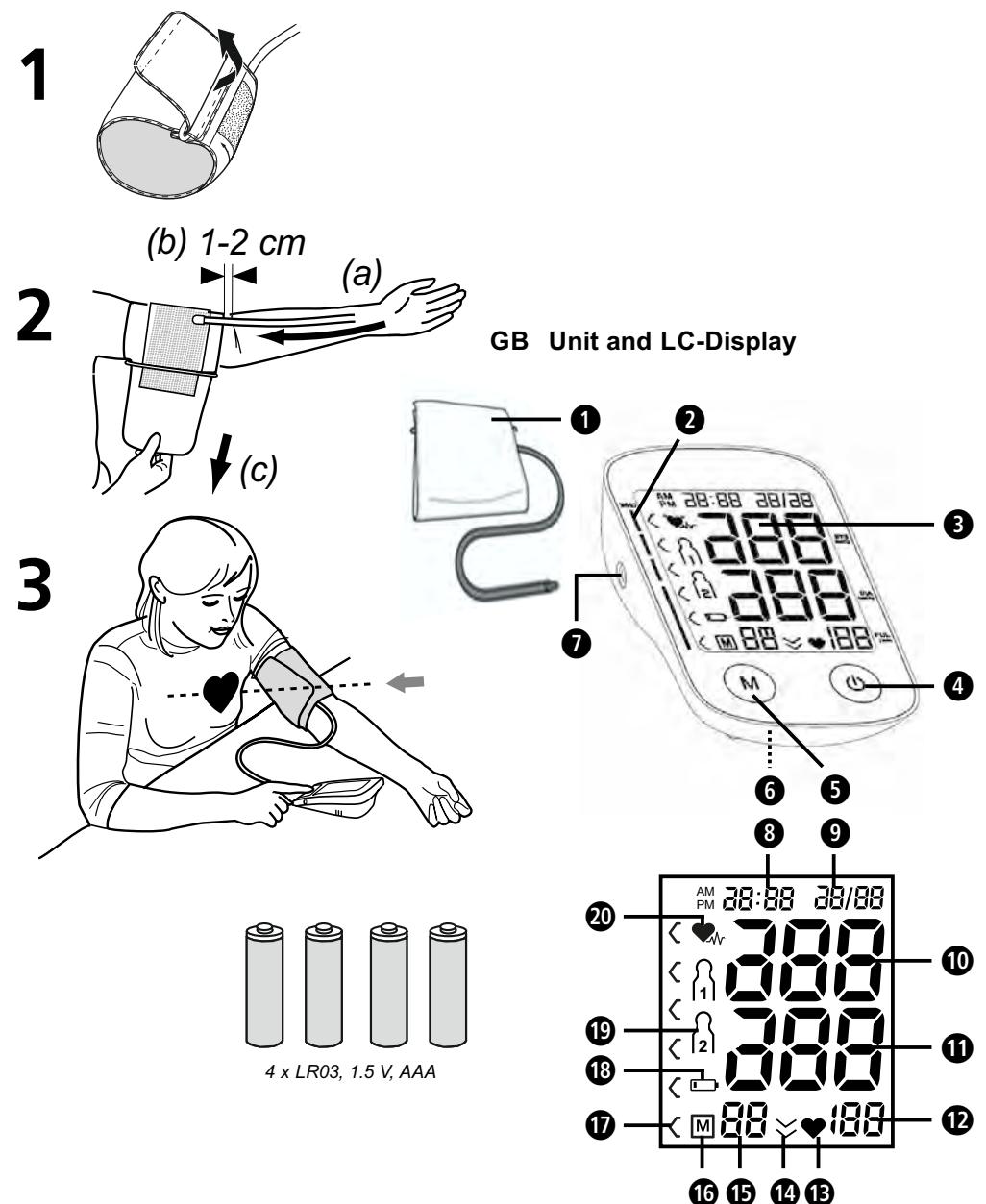
Please read carefully!

### Scope of supply

Please check first of all that the unit is complete. The following parts are included as standard:

- 1 medisana Blood pressure monitor BU 535
- 4 batteries (type AAA, LR03) 1.5V
- 1 cuff with air tube
- 1 storage pouch
- 1 instruction manual

Please contact your supplier if you notice any transport damage on unpacking the unit.

### Explanation of symbols

**IMPORTANT**  
Follow the instructions for use!  
Non-observance of these instructions can result in serious injury or damage to the device.

**WARNING**  
These warning notes must be observed to prevent any injury to the user.

**CAUTION**  
These notes must be observed to prevent any damage to the device.

**NOTE**  
These notes give you useful additional information on the installation or operation.

Device classification: type BF applied part

LOT number

Manufacturer

Date of manufacture

## GB Safety Instructions

Read the instruction manual carefully before using this device, especially the safety instructions, and keep the instruction manual for future use. Should you give this device to another person, it is vital that you also pass on these instructions for use.

- The unit is intended only for use in the home. Consult your doctor prior to using the blood pressure monitor if you are concerned about health matters.
- Only use the unit for its intended purpose in accordance with the instructions for use. Warranty claims become void if the unit is misused.
- Please consult your doctor prior to using the unit if you suffer from illnesses such as arterial occlusive disease.
- The unit may not be used to check the heart rate of a pacemaker.
- Like on any other oscillometric blood pressure measurement devices, certain medical conditions can affect the measurement accuracy, among others: cardiac arrhythmias, low blood pressure, circulatory disorders, states of shock, diabetes, pregnancy, pre-eclampsia etc. Always consult a physician before you use the device.
- This device is not suitable to provide a medical diagnosis. The measurement results are for reference only. Contact your physician to get information about treatment and medicine, which are necessary for you.
- The arrhythmia function does not replace a professional cardiac examination. Check with your doctor if any treatment or medical action is necessary for you.
- Irregularity of pulse or arrhythmia can lead to difficulties in recording a correct reading when measurements are taken using oscillometric blood pressure devices. This device is electronically equipped to detect irregular heartbeat and indicate this with a symbol in the display.
- Please contact your physician about the specific risks of cuff pressure in your individual case.
- If you experience discomfort during a measurement, such as pain in the upper arm or other complaints or in case the device does not stop to inflate the cuff, press the START/STOP button to release the air immediately from the cuff. Loosen the cuff and remove it from your arm.
- Blood pressure measurements in high repetition rates may lead to unwanted side effects, e.g. nerve compression or blood clots.
- Blood pressure measurements - especially if executed frequently - may lead to temporary marks on the skin. These marks may be visible for several days in individual cases. If necessary, contact your physician to get more information.
- This device is not designed to be used by persons with limited physical, sensory or mental abilities, or by persons with insufficient experience and/or knowledge, unless under observation by a person responsible for their safety, or unless they have been instructed in the use of the device.
- This device is intended to be used by adults. The unit is unsuitable for children or babies. Contact your doctor, if you want to use this device on young persons.
- Do not allow children to use the unit. Medical products are not toys!
- The swallowing of small parts like packaging bag, battery, battery cover and so on may cause suffocation.
- The user needs to check that the equipment functions properly and that it is in proper working condition before using.
- If the systolic and diastolic readings seem unusual (too high or too low) on several occasions, despite correct use of the unit, please inform your doctor. This also applies to the rare occasions when an irregular or very weak pulse prevents you from taking readings.
- Do not use the device in moving vehicles.
- The device may not be operated in rooms with high levels of radiation or in the vicinity of high-radiation devices, such as radio transmitters, mobile telephones or microwaves, as this could cause functional defects or incorrect measurements.
- Do not use the equipment where flammable gas (such as anaesthetic gas, oxygen or hydrogen) or flammable liquids (such as alcohol) are present.
- Contact your physician before using the device, if:
  - the cuff should be applied over hurt or burned skin
  - the cuff should be applied on limbs with an inserted catheter or shunt
  - the cuff should be applied on the same side of the body where a mastectomy has been realized
  - a blood pressure measurement should be executed together with other measurements on the same part of the body
  - a control of the blood circulation of the user is necessary
- Never use any other cuff than the one supplied and do not use the cuff together with other persons to avoid infections.
- Do not apply the cuff on any other part of the body than the left upper arm.
- The cuff must not be applied over clothes as this may lead to injuries or inaccurate readings.
- Do not kink or fold the cuff and do not pull it over sharp edges.
- No modification of this unit/device is allowed.
- Do not disassemble the device.
- Avoid high temperatures and direct sunlight.
- Protect the unit against moisture. If liquid does manage to penetrate into the unit, remove the batteries immediately and do not continue to use the unit. In this case contact your specialist dealer or notify us directly.
- Please do not attempt to repair the unit yourself in the event of malfunctions. Only have repairs carried out by authorised service centres.
- If the device has been stored at cool temperatures, you should allow some time to acclimatise at room temperature before using.
- Please do not use a dilution agent, alcohol or petrol to clean the unit.
- Do not hit the device heavily or let it fall down from a high place.
- Do not insert any objects into the device openings and never push the buttons with heavy force or with the help of peak objects. This may lead to damages of the device.
- If the unit is not going to be used for a long period, please remove the batteries.

### SAFETY NOTES FOR BATTERIES

- Do not disassemble batteries!
- Replace all batteries if the unit display shows the low battery symbol.
- Never leave any low battery in the battery compartment since it may leak and cause damage to the unit.
- Increased risk of leakage! Avoid contact with skin, eyes and mucous membranes! If battery acid comes in contact with any of these parts, rinse the affected area with copious amounts of fresh water and seek medical attention immediately!
- If a battery has been swallowed, seek medical attention immediately!
- Replace all of the batteries simultaneously!
- Only replace with batteries of the same type, never use different types of batteries together or used batteries with new ones!
- Insert the batteries correctly, observing the polarity!
- Keep batteries out of children's reach!
- Do not attempt to recharge batteries! There is a danger of explosion!
- Do not short circuit! There is a danger of explosion!
- Do not throw into a fire! There is a danger of explosion!
- Do not throw used batteries into the household refuse; put them in a hazardous waste container or take them to a battery collection point, at the shop where they were purchased!

### Unit and LC-Display

- |                                  |  |
|----------------------------------|--|
| ① Upper arm cuff                 | ② Blood Pressure Indicator (green - yellow - orange - red) |
| ③ LC-display                     | ④ START/STOP button (START/STOP)                           |
| ⑤ M-button (Memory Recall)       | ⑥ Battery compartment (on underside)                       |
| ⑦ Push-in Connector for air tube | ⑧ Display of time  |
| ⑨ Display of date                | ⑩ Display of Systolic Pressure                             |
| ⑪ Display of Diastolic Pressure  | ⑫ Display of Pulse Rate                                    |
| ⑬ Pulse symbol                   | ⑭ Symbol for zero pressure search                          |
| ⑮ Memory Location number         | ⑯ M-symbol (memory)  |
| ⑰ Blood Pressure Indicator       | ⑱ Change Battery symbol                                    |
| ⑲ User memory                    | ⑳ Unit has detected irregular heartbeat                    |

### What is blood pressure?

Blood pressure is the pressure created in the blood vessels by each heart beat. When the heart contracts (= systole) and pumps blood into the arteries, it brings about a rise in pressure. The highest value is known as the systolic pressure and is the first value to be measured. When the cardiac muscle relaxes to take in fresh blood, the pressure in the arteries also falls. Once the arteries are relaxed, a second reading is taken - the diastolic pressure.

### How is blood pressure measured?

The medisana BU 535 is a blood pressure unit which is used to measure blood pressure at the upper arm. The measurement is carried out by a microprocessor, which, via a pressure sensor, measures the vibrations resulting from the inflation and deflation of the cuff over the artery.

### Blood pressure classification

systolic mmHg	diastolic mmHg		Blood Pressure Indicator ② / ⑯
≥ 180	≥ 110	severe hypertension	red
160 - 179	100 - 109	moderate hypertension	orange
140 - 159	90 - 99	mild hypertension	yellow
130 - 139	85 - 89	high-normal blood pressure	green
120 - 129	80 - 84	normal blood pressure	green
< 120	< 80	optimal blood pressure	green

**WARNING**  
Blood pressure that is too low represents just as great a health risk as blood pressure that is too high! Fits of dizziness may lead to dangerous situations arising (e.g. on stairs or in traffic)!

### Influencing and evaluating readings

- Measure your blood pressure several times, then record and compare the results. Do not draw any conclusions from a single reading.
- Your blood pressure readings should always be evaluated by a doctor who is also familiar with your personal medical history. When using the unit regularly and recording the values for your doctor, you should visit the doctor from time to time to keep him updated.
- The user needs to check that the equipment functions properly and that it is in proper working condition before using.
- If the systolic and diastolic readings seem unusual (too high or too low) on several occasions, despite correct use of the unit, please inform your doctor. This also applies to the rare occasions when an irregular or very weak pulse prevents you from taking readings.
- The device needs to be used in accordance to the specified ambient conditions (see "Technical specifications") only. Otherwise the accuracy of readings might be affected.
- Do not use the device in moving vehicles.

### Starting up

#### Insert / replace batteries

You must insert the batteries provided before you can use your unit. The lid of the battery compartment is located on the underside of the unit. Open it by pushing the cover outward and remove it. Insert the 4 AAA LR03 type 1.5V batteries supplied. Ensure correct polarity when inserting the batteries (as marked on the battery compartment cover). Close the battery compartment. Replace the batteries immediately, if the change battery symbol appears on the display or if nothing appears on the display after the unit has been switched on.

#### Settings: Display format, date, time

For a correct operation of the device it is necessary to set time and date. Immediately after inserting the batteries the device is in setting mode for 1 minute. Afterwards, it will switch to standby-mode. You may enter the setting mode at any time by pressing and holding the START/STOP button and the M-button at the same time for approx. 2 seconds. In setting mode, firstly the clock format (12 or 24 hours format) can be set. The value can be adjusted with the M-button . Press the START/STOP button afterwards to enter the date and time setting mode. Adjust the year, month, day, hour and minute figure in sequence. You may adjust the flashing value by pressing the M-button and confirm the value and go to next step by pressing the START/STOP button . After the minutes have been set and the START/STOP button is pressed, all adjustments have been done and the device will switch itself to standby mode again. If no button is pressed during approx. 1 minute during the setting process, the device will automatically switch to standby-mode. If the batteries are changed, the set information is lost and needs to be entered again.

### Fitting the cuff

- Push the end piece of the air tube into the hole on the left side of the unit prior to use.
- Slide the open end of the cuff through the metal bracket so that the Velcro fastener is on the outside and it becomes a cylindrical form (Fig.1). Slide the tube over your left upper arm.
- Position the air hose in the middle of your arm in line with your middle finger (Fig.2) (a). The lower edge of the cuff should be 2 - 3 cm above the crease of the elbow. (b). Pull the cuff tight and close the Velcro fastener (c).
- Measure the pulse on your bare arm.
- Only position the cuff on the right arm if it cannot be used on the left arm. Always carry out measurements on the same arm.
- Replace all of the batteries simultaneously!
- Only replace with batteries of the same type, never use different types of batteries together or used batteries with new ones!
- Insert the batteries correctly, observing the polarity!
- Keep batteries out of children's reach!
- Do not attempt to recharge batteries! There is a danger of explosion!
- Do not short circuit! There is a danger of explosion!

### Taking a blood pressure measurement

- After the cuff has been appropriately positioned, the measurement can begin:
- Switch the unit on by pressing the START/STOP button . The display will shortly show all available symbols.
  - The medisana blood pressure monitor BU 535 gives you the opportunity to assign readings to one of two user memories, each of which provides 120 memory slots. After switching on the device, press the M-button to choose between user 1 and user 2. If no selection is made within 5 seconds, the following measurement will be assigned to the current shown user.
  - The last value measured for this user is shown in the display (if no value is available, "0" is shown). The device now starts to deflate the cuff completely - the symbol for zero pressure search is shown in the display.

- The unit is now ready for measurement and it automatically inflates the cuff slowly in order to measure your blood pressure. The rising pressure in the cuff is shown on the display. The unit inflates the cuff until sufficient pressure has built up for a measurement. Then the unit slowly releases air from the cuff and carries out the measurement. When the device detects the signal, the heart symbol starts to flash on the display.
- When the measurement is finished, the cuff is deflated completely. The systolic and diastolic blood pressure and the pulse value appear on the display . The blood pressure indicator is shown next to the relevant coloured bar depending on the blood pressure classification. If the unit has detected an irregular heartbeat, the display will also be shown.

- WARNING**  
Do not take any therapeutic measures on the basis of a self measurement. Never alter the dose of a medicine prescribed by a doctor.

- The readings are automatically saved in the selected memory ( or ). Up to 120 measured values with date and time can be stored in each memory along with time and date.

- The measurement readings remain on the display. The unit switches off automatically after approx. 1 minute if no buttons are pressed. The unit may also be switched off using the START/STOP button .

### Discontinuing a measurement

If it is necessary to interrupt a blood pressure measurement for any reason (e.g. the patient feels uncomfortable), the START/STOP button can be pressed at any time. The device immediately decreases the cuff pressure automatically.

### Display stored values

This unit features 2 separate memories, each with a capacity of 120 memory slots. Results are automatically stored in the memory selected by the user. To call up the measured values stored, press the M-button when the device is switched off to activate the memory recall mode. A user memory ( or ) along with the number of saved values is shown. Change the user memory with the START/STOP button . Press the M-button to show the average values of the last 3 measurements of this user (along with ). With each press on the M-button the display proceeds to show the average values for all measurements of this user memory ("AL"). The average value of the measurement results of the last 7 days taken between 5 o'clock and 9 o'clock in the morning ("AM") and the average value of the measurement results of the last 7 days taken between 6 o'clock and 8 o'clock in the evening ("PM"). Now press the M-button again to display the last measurement value of the user. Repeatedly pressing the M-button displays the respective values measured previously. If no further values are saved, the display will show "0". If no button is pressed, the unit automatically switches itself off after approximately 1 minute. Memory recall mode may also be exited at any time by pressing the START/STOP button , which will also switch the unit off. The oldest value is deleted if 120 measured values have been stored in the memory and a new value is saved.

### Delete saved values

If you want to delete the saved measurements of a user, first follow the instructions under "Display stored values" to display any record of the user. Now press and hold the M-button for approx. 3 seconds. The display will show "CL" and "00" and the values are deleted.

### Error messages and error remedying

The following notifications in the display indicate problems or errors:

Notification	Cause and correction
Hi or Lo	The measurement result is out of the measurable range. Repeat the measurement and contact your physician if necessary.
Unusual measurement results	Cuff not fitted properly; Movement or speaking during measurement; Incorrect body posture; Heart arrhythmia. Repeat the measurement and contact your physician if necessary.
	Replace all batteries.
Er 0 / Er 1 / Er 2	Pressure system unstable at start of the measurement / Error in measuring the systolic pressure / Error in measuring the diastolic pressure. <b>Do not move!</b>
Er 3 / Er 4	Pressure system is blocked or sleeve is too tight during inflation / Pressure system has a leak or sleeve is too loose during inflation <b>Put the sleeve on correctly!</b>
Er 5 / Er 6 / Er 7 / Er 8 / Er A	Sleeve pressure above 300 mmHg / Longer than 160 seconds with sleeve pressure over 15 mmHg / EEPROM access error. Error when testing the functions on the device / Failure of pressure sensor <b>Perform measurement again after 5 minutes!</b>
No reaction after pressing a button or insertion of batteries	Remove the batteries, wait for 5 minutes and reinsert the batteries.

Please contact the manufacturer resp. the service centre if you can't solve the problem. Do not disassemble the unit by yourself.

### Cleaning and care

Remove the batteries before cleaning the unit. Clean the unit and the cuff using a soft cloth lightly moistened

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