Instruction Manual

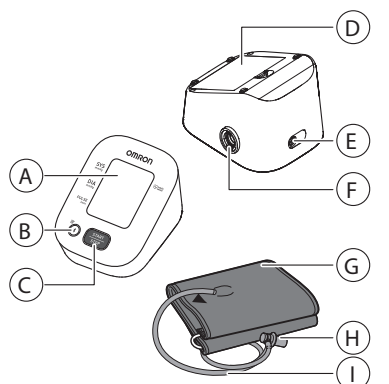
Automatic Upper Arm Blood Pressure Monitor M2 Intelli IT (HEM-7143T1-EBK)

English

Read Instruction manual 1 and 2 before use.

For symbols information, refer to "Symbols Description" on the back side of the another "Instruction manual 1".

All for Healthcare

Introduction

Thank you for purchasing the M2 Intelli IT Automatic Upper Arm Blood Pressure Monitor. This blood pressure monitor uses the oscillometric cuff method of blood pressure measurement. This monitor detects your blood movement through your brachial artery and converts the movement into a digital reading.

Safety Instructions
This instruction manual provides you with important information about the M2 Intelli IT Automatic Upper Arm Blood Pressure Monitor. To ensure the safe and proper use of this monitor, READ and UNDERSTAND all of these instructions. If you do not understand these instructions or have any questions, contact your OMRON retail outlet or distributor before attempting to use this monitor. For specific information about your own blood pressure, consult with your physician.

Intended Use
This device is a digital monitor intended for use in measuring blood pressure and pulse rate in adult patient population. This monitor detects the appearance of irregular heartbeats during measurement and indicates this via a symbol with readings. It is mainly designed for general household use.

Receiving and Inspection
Remove this monitor from the packaging and inspect for damage. If this monitor is damaged, DO NOT USE and contact your OMRON retail outlet or distributor.

Important Safety Information

Read the Important Safety Information in this instruction manual before using this device. For specific information about your own blood pressure, CONSULT WITH YOUR PHYSICIAN.

Warning
Indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

- DO NOT use this monitor on infants, toddlers, children or people who cannot express themselves.
- DO NOT adjust medication based on readings from this blood pressure monitor. Take medication as prescribed by your physician. If a physician is qualified to diagnose and treat high blood pressure.
- DO NOT use this monitor on an injured arm or an arm under medical treatment.
- DO NOT use the arm cuff on your arm while on an antihypertensive drug.
- DO NOT use this monitor in areas containing high frequency electrical equipment (e.g., magnetic resonance imaging (MRI) equipment, computerized tomography (CT) scanners). This may result in incorrect operation of the monitor and/or cause an injury.
- DO NOT use this monitor in oxygen rich environments or near flammable gases.
- Consult with your physician before using this monitor if you have any arrhythmias such as atrial or ventricular premature beats or atrial fibrillation, arterial sclerosis, poor peripheral circulation, pregnancy, pre-eclampsia or renal disease. NOTE that in these conditions, blood pressure to patient motion, trembling, or shivering may affect the measurement reading.
- NEVER diagnose or treat yourself based on your readings.
- DO NOT use any medication while you are taking a measurement. If you have any questions, consult with your physician.
- DO NOT use in or unplug the AC adapter from the electric power outlet.
- DO NOT disassemble or attempt to repair the AC adapter.

Battery Handling and Usage
DO NOT insert batteries or your polarities incorrectly aligned.
ONLY use "AA" alkaline or manganese batteries in this monitor. DO NOT use other types of batteries. DO NOT use new and used batteries together. DO NOT use different brands of batteries.
Remove batteries if this monitor will not be used for a long period of time.
Battery fluid should get in your eyes, immediately rinse with plenty of clean water. Consult with your physician immediately.
If battery fluid should get on your skin, wash your skin immediately with plenty of clean water.
If irritation, injury or pain persists, consult with your physician.
DO NOT use batteries after their expiration date.
Periodically check batteries to ensure they are in good working condition.

General Precautions

When you take a measurement on the right arm, the air tube should be on the right side of your elbow. Be careful not to rest your arm on the air tube.

Caution
Blood pressure may differ between the right and left arms, and may require a different measurement value. Always use the same arm for measurements. If the values between both arms differ significantly, check with your physician on the next use.
This product contains small parts that may cause a choking hazard if swallowed by infants, toddlers and children.

Battery Handling and Usage
DO NOT use the AC adapter if this monitor or the AC adapter is damaged. If the AC adapter was present, remove it from the power and unplug the AC adapter immediately.
DO NOT use the AC adapter into the appropriate voltage outlet.
DO NOT use the AC adapter in areas containing high frequency electrical equipment (e.g., magnetic resonance imaging (MRI) equipment, computerized tomography (CT) scanners). This may result in incorrect operation of the monitor and/or cause an injury.

Caution
Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient, or cause damage to the equipment.

DO NOT use this monitor on an arm where intravenous access or therapy, or an arteriovenous (A-V) shunt, is present because of potential interference to blood flow and could result in injury.
DO NOT use in or unplug the AC adapter from the electric power outlet.
DO NOT disassemble or attempt to repair the AC adapter.

Battery Handling and Usage
DO NOT use the AC adapter if this monitor or the AC adapter is damaged. If the AC adapter was present, remove it from the power and unplug the AC adapter immediately.
DO NOT use the AC adapter into the appropriate voltage outlet.
DO NOT use the AC adapter in areas containing high frequency electrical equipment (e.g., magnetic resonance imaging (MRI) equipment, computerized tomography (CT) scanners). This may result in incorrect operation of the monitor and/or cause an injury.

Caution
Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient, or cause damage to the equipment.

DO NOT use this monitor on an arm where intravenous access or therapy, or an arteriovenous (A-V) shunt, is present because of potential interference to blood flow and could result in injury.
DO NOT use in or unplug the AC adapter from the electric power outlet.
DO NOT disassemble or attempt to repair the AC adapter.

Battery Handling and Usage
DO NOT use the AC adapter if this monitor or the AC adapter is damaged. If the AC adapter was present, remove it from the power and unplug the AC adapter immediately.
DO NOT use the AC adapter into the appropriate voltage outlet.
DO NOT use the AC adapter in areas containing high frequency electrical equipment (e.g., magnetic resonance imaging (MRI) equipment, computerized tomography (CT) scanners). This may result in incorrect operation of the monitor and/or cause an injury.

Caution
Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient, or cause damage to the equipment.

DO NOT use this monitor on an arm where intravenous access or therapy, or an arteriovenous (A-V) shunt, is present because of potential interference to blood flow and could result in injury.
DO NOT use in or unplug the AC adapter from the electric power outlet.
DO NOT disassemble or attempt to repair the AC adapter.

Battery Handling and Usage
DO NOT use the AC adapter if this monitor or the AC adapter is damaged. If the AC adapter was present, remove it from the power and unplug the AC adapter immediately.
DO NOT use the AC adapter into the appropriate voltage outlet.
DO NOT use the AC adapter in areas containing high frequency electrical equipment (e.g., magnetic resonance imaging (MRI) equipment, computerized tomography (CT) scanners). This may result in incorrect operation of the monitor and/or cause an injury.

Caution
Indicates a potentially hazardous situation which, if not avoided, may result in minor or moderate injury to the user or patient, or cause damage to the equipment.

OMRON

IMI-HEM-7143T1-EBK-El-02-04-2022

Display/Problem	Possible Cause	Solution
ES appears		Apply the arm cuff correctly, then take another measurement. Refer to section 6 of instruction manual 1. The pulse rate is not detected correctly.
E1 appears		Press the [START/STOP] button again. If the "E1" symbol continues to appear, we recommend you consult with your physician.
E2 appears		Press the [START/STOP] button again. If the "E2" symbol continues to appear, we recommend you consult with your physician.
E3 appears		Follow the instructions shown in the "OMRON Connect" app. If the "E3" symbol still appears, check the app, contact your OMRON retail outlet or distributor.
E4 appears		Follow the instructions shown in the "OMRON Connect" app. If the "E4" symbol still appears, check the app, contact your OMRON retail outlet or distributor.
E5 appears		Refer to section 5 of instruction manual 2 for pairing your monitor with your smart device, or press the [START/STOP] button to cancel pairing off.
E6 appears		More than 24 readings are not transferred.
E7 appears		Your monitor is not paired or not connected to your smart device.
E8 appears		There are 30 readings in memory to be transferred.
E9 appears		Batteries are low.
E10 appears		Batteries are depleted.
E11 appears		Check the battery installation for proper installation of section 3 of instruction manual 2.
E12 appears		Battery polarities are not properly aligned.
E13 appears		Blood pressure varies constantly. Many factors including stress, time of day, and/or how you apply the arm cuff, may affect your blood pressure. Review section 4 of instruction manual 2.
E14 appears		Follow the instructions shown in the smart device, or visit the "Help" section in the OMRON Connect app. If the problem still persists, contact your OMRON retail outlet or distributor.
E15 appears		Any other communication issue occurs.
E16 appears		Press the [START/STOP] button to turn the monitor off. Wait for 30 seconds. Then re-install batteries.
E17 appears		When you take a measurement, contact your OMRON retail outlet or distributor.

Display/Problem	Possible Cause	Solution
E18 appears		Replacing all 4 batteries with new ones is recommended. Refer to section 3 of instruction manual 2.
E19 appears		Batteries are low.
E20 appears		Check the battery installation for proper installation of section 3 of instruction manual 2.
E21 appears		Battery polarities are not properly aligned.
E22 appears		Blood pressure varies constantly. Many factors including stress, time of day, and/or how you apply the arm cuff, may affect your blood pressure. Review section 4 of instruction manual 2.
E23 appears		Follow the instructions shown in the smart device, or visit the "Help" section in the OMRON Connect app. If the problem still persists, contact your OMRON retail outlet or distributor.
E24 appears		Any other communication issue occurs.
E25 appears		Press the [START/STOP] button to turn the monitor off. Wait for 30 seconds. Then re-install batteries.
E26 appears		When you take a measurement, contact your OMRON retail outlet or distributor.

Display/Problem	Possible Cause	Solution
E27 appears		Nothing appears on the display of the monitor.
E28 appears		Battery polarities are not properly aligned.
E29 appears		Blood pressure varies constantly. Many factors including stress, time of day, and/or how you apply the arm cuff, may affect your blood pressure. Review section 4 of instruction manual 2.
E30 appears		Follow the instructions shown in the smart device, or visit the "Help" section in the OMRON Connect app. If the problem still persists, contact your OMRON retail outlet or distributor.
E31 appears		Any other communication issue occurs.
E32 appears		Press the [START/STOP] button to turn the monitor off. Wait for 30 seconds. Then re-install batteries.
E33 appears		When you take a measurement, contact your OMRON retail outlet or distributor.

Display/Problem	Possible Cause	Solution
E34 appears		Nothing appears on the display of the monitor.
E35 appears		Battery polarities are not properly aligned.
E36 appears		Blood pressure varies constantly. Many factors including stress, time of day, and/or how you apply the arm cuff, may affect your blood pressure. Review section 4 of instruction manual 2.
E37 appears		Follow the instructions shown in the smart device, or visit the "Help" section in the OMRON Connect app. If the problem still persists, contact your OMRON retail outlet or distributor.
E38 appears		Any other communication issue occurs.
E39 appears		Press the [START/STOP] button to turn the monitor off. Wait for 30 seconds. Then re-install batteries.
E40 appears		When you take a measurement, contact your OMRON retail outlet or distributor.

Display/Problem	Possible Cause	Solution
E41 appears		Nothing appears on the display of the monitor.
E42 appears		Battery polarities are not properly aligned.
E43 appears		Blood pressure varies constantly. Many factors including stress, time of day, and/or how you apply the arm cuff, may affect your blood pressure. Review section 4 of instruction manual 2.
E44 appears		Follow the instructions shown in the smart device, or visit the "Help" section in the OMRON Connect app. If the problem still persists, contact your OMRON retail outlet or distributor.
E45 appears		Any other communication issue occurs.
E46 appears		Press the [START/STOP] button to turn the monitor off. Wait for 30 seconds. Then re-install batteries.
E47 appears		When you take a measurement, contact your OMRON retail outlet or distributor.

Display/Problem	Possible Cause	Solution
E48 appears		Nothing appears on the display of the monitor.
E49 appears		Battery polarities are not properly aligned.
E50 appears		Blood pressure varies constantly. Many factors including stress, time of day, and/or how you apply the arm cuff, may affect your blood pressure. Review section 4 of instruction manual 2.
E51 appears		Follow the instructions shown in the smart device, or visit the "Help" section in the OMRON Connect app. If the problem still persists, contact your OMRON retail outlet or distributor.
E52 appears		Any other communication issue occurs.
E53 appears		Press the [START/STOP] button to turn the monitor off. Wait for 30 seconds. Then re-install batteries.
E54 appears		When you take a measurement, contact your OMRON retail outlet or distributor.

Display/Problem	Possible Cause	Solution
E55 appears		Nothing appears on the display of the monitor.
E56 appears		Battery polarities are not properly aligned.
E57 appears		Blood pressure varies constantly. Many factors including stress, time of day, and/or how you apply the arm cuff, may affect your blood pressure. Review section 4 of instruction manual 2.
E58 appears		Follow the instructions shown in the smart device, or visit the "Help" section in the OMRON Connect app. If the problem still persists, contact your OMRON retail outlet or distributor.
E59 appears		Any other communication issue occurs.
E60 appears		Press the [START/STOP] button to turn the monitor off. Wait for 30 seconds. Then re-install batteries.
E61 appears		When you take a measurement, contact your OMRON retail outlet or distributor.

Display/Problem	Possible Cause	Solution
E62 appears		Nothing appears on the display of the monitor.
E63 appears		Battery polarities are not properly aligned.
E64 appears		Blood pressure varies constantly. Many factors including stress, time of day, and/or how you apply the arm cuff, may affect your blood pressure. Review section 4 of instruction manual 2.
E65 appears		Follow the instructions shown in the smart device, or visit the "Help" section in the OMRON Connect app. If the problem still persists, contact your OMRON retail outlet or distributor.
E66 appears		Any other communication issue occurs.
E67 appears		Press the [START/STOP] button to turn the monitor off. Wait for 30 seconds. Then re-install batteries.
E68 appears		When you take a measurement, contact your OMRON retail outlet or distributor.

Display/Problem	Possible Cause	Solution
E69 appears		Nothing appears on the display of the monitor.
E70 appears		Battery polarities are not properly aligned.
E71 appears		Blood pressure varies constantly. Many factors including stress, time of day, and/or how you apply the arm cuff, may affect your blood pressure. Review section 4 of instruction manual 2.
E72 appears		Follow the instructions shown in the smart device, or visit the "Help" section in the OMRON Connect app. If the problem still persists, contact your OMRON retail outlet or distributor.
E73 appears		Any other communication issue occurs.
E74 appears		Press the [START/STOP] button to turn the monitor off. Wait for 30 seconds. Then re-install batteries.
E75 appears		When you take a measurement, contact your OMRON retail outlet or distributor.

Display/Problem	Possible Cause	Solution
E76 appears		Nothing appears on the display of the monitor.
E77 appears		Battery polarities are not properly aligned.
E78 appears		Blood pressure varies constantly. Many factors including stress, time of day, and/or how you apply the arm cuff, may affect your blood pressure. Review section 4 of instruction manual 2.
E79 appears		Follow the instructions shown in the smart device, or visit the "Help" section in the OMRON Connect app. If the problem still persists, contact your OMRON retail outlet or distributor.
E80 appears		Any other communication issue occurs.
E81 appears		Press the [START/STOP] button to turn the monitor off. Wait for 30 seconds. Then re-install batteries.
E82 appears		When you take a measurement, contact your OMRON retail outlet or distributor.

Display/Problem	Possible Cause	Solution
E83 appears		Nothing appears on the display of the monitor.
E84 appears		Battery polarities are not properly aligned.
E85 appears		Blood pressure varies constantly. Many factors including stress, time of day, and/or how you apply the arm cuff, may affect your blood pressure. Review section 4 of instruction manual 2.
E86 appears		Follow the instructions shown in the smart device, or visit the "Help" section in the OMRON Connect app. If the problem still persists, contact your OMRON retail outlet or distributor.
E87 appears		Any other communication issue occurs.
E88 appears		Press the [START/STOP] button to turn the monitor off. Wait for 30 seconds. Then re-install batteries.
E89 appears		When you take a measurement, contact your OMRON retail outlet or distributor.

Display/Problem	Possible Cause	Solution
E90 appears		Nothing appears on the display of the monitor.
E91 appears		Battery polarities are not properly aligned.
E92 appears		Blood pressure varies constantly. Many factors including stress, time of day, and/or how you apply the arm cuff, may affect your blood pressure. Review section 4 of instruction manual 2.
E93 appears		Follow the instructions shown in the smart device, or visit the "Help" section in the OMRON Connect app. If the problem still persists, contact your OMRON retail outlet or distributor.
E94 appears		Any other communication issue occurs.
E95 appears		Press the [START/STOP] button to turn the monitor off. Wait for 30 seconds. Then re-install batteries.
E96 appears		When you take a measurement, contact your OMRON retail outlet or distributor.

Display/Problem	Possible Cause	Solution
E97 appears		Nothing appears on the display of the monitor.
E98 appears		Battery polarities are not properly aligned.
E99 appears		Blood pressure varies constantly. Many factors including stress, time of day, and/or how you apply the arm cuff, may affect your blood pressure. Review section 4 of instruction manual 2.
E100 appears		Follow the instructions shown in the smart device, or visit the "Help" section in the OMRON Connect app. If the problem still persists, contact your OMRON retail outlet or distributor.
E101 appears		Any other communication issue occurs.
E102 appears		Press the [START/STOP] button to turn the monitor off. Wait for 30 seconds. Then re-install batteries.
E103 appears		When you take a measurement, contact your OMRON retail outlet or distributor.

Display/Problem	Possible Cause	Solution
E104 appears		Nothing appears on the display of the monitor.
E105 appears		Battery polarities are not properly aligned.
E106 appears		Blood pressure varies constantly. Many factors including stress, time of day, and/or how you apply the arm cuff, may affect your blood pressure. Review section 4 of instruction manual 2.
E107 appears		Follow the instructions shown in the smart device, or visit the "Help" section in the OMRON Connect app. If the problem still persists, contact your OMRON retail outlet or distributor.
E108 appears		Any other communication issue occurs.
E109 appears		Press the [START/STOP] button to turn the monitor off. Wait for 30 seconds. Then re-install batteries.
E110 appears		When you take a measurement, contact your OMRON retail outlet or distributor.

Display/Problem	Possible Cause	Solution
E111 appears		Nothing appears on the display of the monitor.
E112 appears		Battery polarities are not properly aligned.
E113 appears		Blood pressure varies constantly. Many factors including stress, time of day, and/or how you apply the arm cuff, may affect your blood pressure. Review section 4 of instruction manual 2.
E114 appears		Follow the instructions shown in the smart device, or visit the "Help" section in the OMRON Connect app. If the problem still persists, contact your OMRON retail outlet or distributor.
E115 appears		Any other communication issue occurs.
E116 appears		Press the [START/STOP] button to turn the monitor off. Wait for 30 seconds. Then re-install batteries.
E117 appears		When you take a measurement, contact your OMRON retail outlet or distributor.

Display/Problem	Possible Cause	Solution
E118 appears		Nothing appears on the display of the monitor.
E119 appears		Battery polarities are not properly aligned.
E120 appears		Blood pressure varies constantly. Many factors including stress, time of day, and/or how you apply the arm cuff, may affect your blood pressure. Review section 4 of instruction manual 2.
E121 appears		Follow the instructions shown in the smart device, or visit the "Help" section in the OMRON Connect app. If the problem still persists, contact your OMRON retail outlet or distributor.
E122 appears		Any other communication issue occurs.
E123 appears		Press the [START/STOP] button to turn the monitor off. Wait for 30 seconds. Then re-install batteries.
E124 appears		When you take a measurement, contact your OMRON retail outlet or distributor.

IP Classification	Monitor: IP20 Polarities: Inverter adapter: IP21 for HEM-CMO, IP22 for HEM-BFH01
Rating	DCV 6 4 W
Power source	4 "AA" batteries, 1.5 V or optional rechargeable NiMH (EN ISO 14608-2) 240V 50-/60 Hz D.I.2 - 0.065 A
Battery life	Approximately 900 measurements (using new alkaline batteries) and including arm cuff. Depending on the type of battery and arm cuff.
Durable period (Service life)	Monitor: 5 years Arm cuff: 2 years Optional AC adapter: 5 years
Operating conditions	+10 to +40°C / 15 to 100% RH (non-condensing) 800 to 1060 hPa
Storage / Transport conditions	-20 to +40°C / 10 to 90% RH (non-condensing) 800 to 1060 hPa
Weight	Monitor: approximately 250 g (not including battery) Arm cuff: approximately 110 g
Dimensions (approximately)	Monitor: 103 mm (W) x 82 mm (H) x 140 mm (D) Arm cuff: 146 mm x 466 mm x 61 mm
Cuff circumference applicable	17 to 42 cm (included arm cuff: 16 to 42 cm)
Memory	Stores up to 30 readings
Contents	Monitor, arm cuff (HEM-CR24), 4 "AA" batteries, storage case Internally powered ME equipment (when using only batteries) Arm cuff (HEM-CR24) optional AC adapter
Protection against electric shock	Type BF (arm cuff)
Applied part	Type BF (arm cuff)

Display/Problem	Possible Cause	Solution
E125 appears		Nothing appears on the display of the monitor.
E126 appears		Battery polarities are not properly aligned.
E127 appears		Blood pressure varies constantly. Many factors including stress, time of day, and/or how you apply the arm cuff, may affect your blood pressure.

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